

I N S T R U C T I O N S



Extra Wide Tall-Ette® Toilet Seat with Arms and Legs

CATALOG NUMBER 72588-1000

1. Insert the two safety bars into the four holes in the top of the plastic seat.

2. Push the four rubber feet onto the ends of the safety bars.

(Note: Dipping the rubber feet in soapy water will make them slide on easier.)

3. Turn the seat over and place the Lok-In-EI® Bracket on the seat by pressing the two large pins into the two wells provided. Turn seat right side up. The flat bar on the Lok-In-EI® Bracket will extend from the rear of the seat.

4. Lift the toilet bowl seat to the raised position.

5. Slip the flat bar of the Lok-In-EI® Bracket under the hinge bar of the toilet seat. The bottom ring of the elevated seat must fit inside the toilet bowl.

6. Adjust the legs so the seat sits firmly on the bowl and the legs rest on the floor.

7. The seat is now ready for use. It may be cleaned by using a mild detergent solution and sterilized with alcohol.



Caution:

Do not use this product without the Lok-In-EI® Bracket.

Do not use this product if the seat does not fit inside the toilet bowl rim.

Maddak Inc.

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email custservice@maddak.com • www.maddak.com